



# Peroxi-DS™

## DISINFECTANT & SANITIZER

FOR HOSPITALS, HOUSEHOLD, INSTITUTIONAL, COMMERCIAL AND INDUSTRIAL USE. USE THIS PRODUCT ON HARD, NON-POROUS SURFACES IN: HOTELS, MOTELS, MEDICAL OFFICES, CAFETERIAS, SCHOOLS, DAY CARE CENTERS, NURSERIES.

### ACTIVE INGREDIENT:

Hydrogen Peroxide: 01.22%

EPA Reg. No. 6836-385-41567 EPA Est. No. 41567-NC-01

### KEEP OUT OF REACH OF CHILDREN

This product is a proven "one-step" disinfectant - cleaner - virucide\* - which is effective in the presence of 5% serum contamination in 3 minute contact time. This product is a 3 minute acting disinfectant and a 1 minute sanitizer<sup>1</sup>. This product is a one-step hospital disinfectant, cleaner that is effective against a broad spectrum of bacteria and is virucidal\*. It inhibits the growth of mold and mildew and removes soap scum. Use this product on washable hard, non-porous, non-food contact surfaces of: Bathroom fixtures, cabinets, chairs, counters, desks, doorknobs, floors, showers, sinks, tables, toilet bowl surfaces, urinal exteriors, walls, and other hard non-porous surfaces made of: glazed ceramic, glazed porcelain, sealed stone, sealed terrazzo, stainless steel.

This product is effective as a disinfectant against: Pseudomonas aeruginosa, Salmonella enterica, Staphylococcus aureus, Enterobacter aerogenes, Escherichia coli, Klebsiella pneumoniae, Methicillin Resistant Staphylococcus aureus, \*Herpes Simplex Type 2, \*HIV-1, \*Influenza A/ Hong Kong, \*Norovirus.

This product is a sanitizer for hard, non-porous kitchen, bathroom and household non-food contact surfaces: Enterobacter aerogenes, Staphylococcus aureus.

Si no sabes el inglés, solicite instrucciones a su supervisor antes de usar el product.

Product Code	627RTU	627MP
Volume	32 US ounces	1.06 US gallons
Shipping Package	12 x 32 ounces	4 x 1.06 US gallons
Pallet Quantity	44	48

627 DIRECTIONS 08/20



# Peroxi-DS™

## DISINFECTANT & SANITIZER

### DIRECTIONS

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

This product is not for use on medical device surfaces.

#### GENERAL CLEANING/DEODORIZING DIRECTIONS:

Spray soiled area then wipe clean. Repeat for heavily soiled areas.

#### DISINFECTION/\*VIRUCIDAL DIRECTIONS:

Spray 6 - 8 inches from surface, until surfaces are thoroughly wet. Do not breathe spray. Treated surfaces must remain wet for 3 minutes. Wipe dry with a clean cloth, sponge, mop or towel or allow to air dry.

#### NON-ACID TOILET BOWL AND URINAL DIRECTIONS:

Apply this product to exposed surfaces including under the rim with a sprayer. Use a toilet brush to scrub the bowl. Repeat and allow to stand for 1 minute, then flush.

#### MOLD AND MILDEW CONTROL DIRECTIONS:

This product effectively inhibits the growth of mold and mildew, and the odors caused by them, when applied to hard, non-porous surfaces. Spray 6 - 8 inches from surface, until surfaces are thoroughly wet. Do not breath spray. Allow to air dry on surface. Repeat every 7 days or when mold or mildew growth returns.

**\*KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES<sup>1</sup>/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1).

#### SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.

**PERSONAL PROTECTION:** Clean-up must always be done wearing protective latex gloves, gowns, masks and eye protection. **CLEANING PROCEDURE:** Blood and other body fluids containing HIV-1 must be thoroughly cleaned from surfaces and objects before application of this product. **CONTACT TIME:** Leave surface wet for 3 minutes. **DISPOSAL OF INFECTIOUS MATERIALS:** Blood, body fluids, cleaning materials and clothing must be autoclaved and disposed of according to local regulations for infectious waste disposal.

#### TO SANITIZE AND DEODORIZE NON-FOOD CONTACT SURFACES:

Spray this product until thoroughly wet. Treated surfaces must remain wet for 1 minute. Wipe dry with a clean cloth, sponge, mop or towel, or allow to air dry. For heavily soiled areas, pre-cleaning is required.

Product Code	627RTU	627MP
Volume	32 US ounces	1.06 US gallons
Shipping Package	12 x 32 ounces	4 x 1.06 US gallons
Pallet Quantity	44	48



# Peroxi-DS™

## DISINFECTANT & SANITIZER

---

### STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal.

#### PESTICIDE STORAGE:

Open dumping is prohibited. Store in original container in areas inaccessible to children.

#### CONTAINER DISPOSAL:

Nonrefillable container. Do not reuse or refill this container. Triple rinse. Offer for recycling or reconditioning or puncture and dispose of in a sanitary landfill, or incineration, or if allowed by state and local authorities, by burning. If burned stay out of smoke.

<sup>1</sup>Non-food contact surfaces

Product Code	627RTU	627MP
Volume	32 US ounces	1.06 US gallons
Shipping Package	12 x 32 ounces	4 x 1.06 US gallons
Pallet Quantity	44	48